


How to Interpret Results

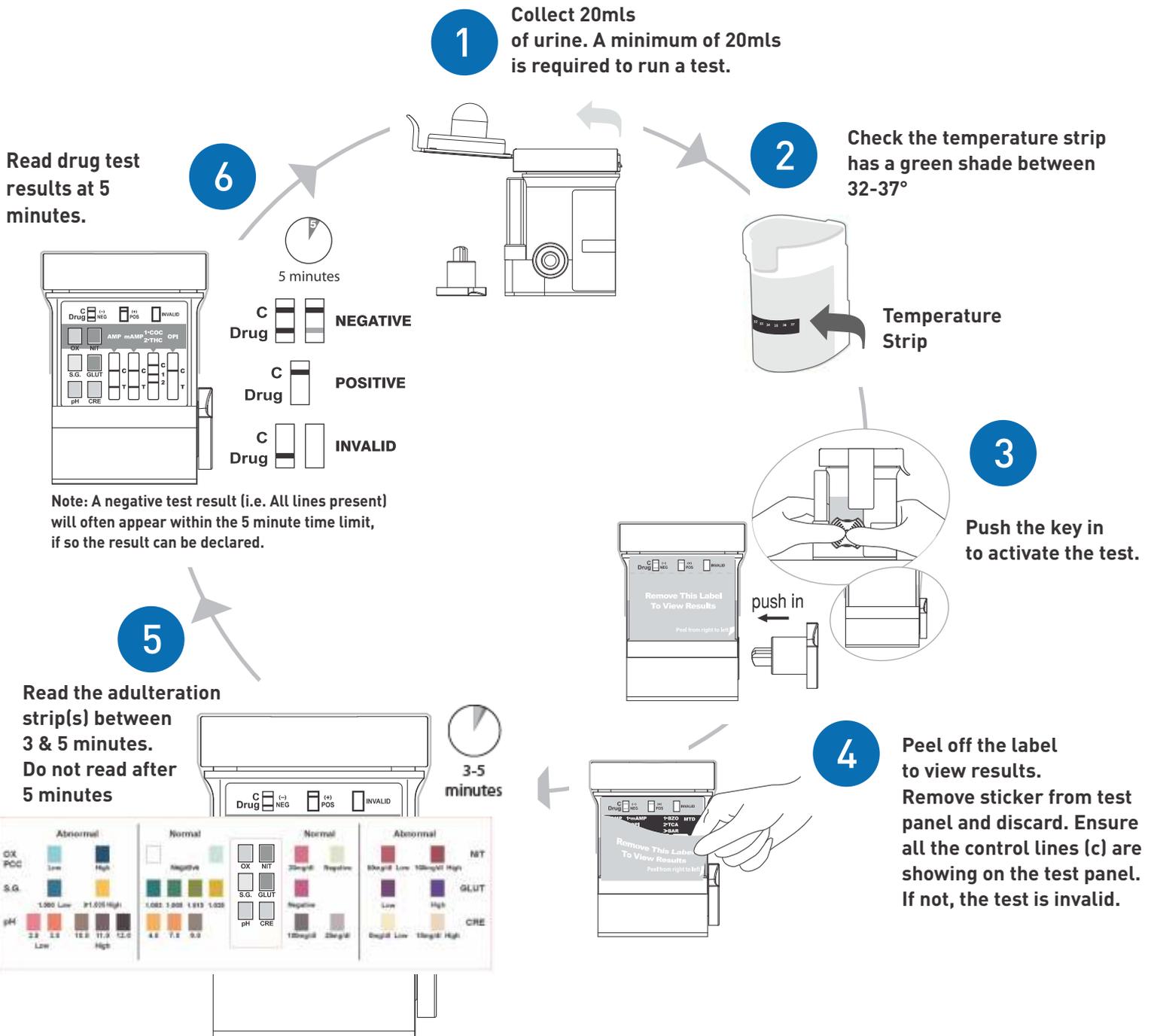

Sober Check - The leader in certified drug & alcohol testing equipment

Procedure for Medix Pro-Split Cup

step 1

- Remove the Medix cup from foil package.
- Remove key from the Medix cup and open the lid.

step 2

- Collect a minimum of 20mls of urine.
- Check temperature strip - any faint change in colour on the temperature strip indicates that the urine is at body temperature - between 32 - 37 ° C.

step 3

- Push key in to activate test.
- Remove green sticker from test panel and start timing the test.

step 4

- Wait for 3-5 minutes then read adulteration results.

step 5

- Place adulterant colour comparison chart window over the test panel on the Medix cup.
- Compare the colours on the chart to the colours on the cup to ensure that they fall within the normal range.
- If the adulterants are abnormal this indicates the urine has been altered or contaminated and a new sample should be obtained for further testing.

step 6

- Read drug test results at 5 minutes.
- Ensure all the control lines (C) are showing on the test panel, if not the test is invalid.
- A negative test will show a line, even if it is a faint line, this is still a negative.

step 7

- Any positive results will have no line present; these tests should be then sent to a lab for confirmation testing.

For more information or help on reading a test result contact Sober Check Ltd on 0800 700 777.