


# Procedure for Synthetic Cannabis Testing

## Three Easy Steps

1. Remove the card test from the sealed pouch. Write donor name or ID on the plastic. Remove the cap to expose the sampling tips.
2. Immerse the sampling tip into the urine specimen for approximately 15 to 30 seconds. Do not allow specimen to come in contact with the plastic housing. Replace the cap over the sampling tip and then place the test card on a flat surface.
3. Read results of drugs of abuse tests in 5 minutes. Do not interpret result after 60 minutes. Refer to interpretation of results section.

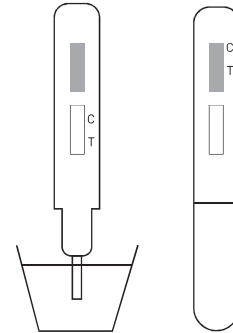


## Interpretation of Results

**Negative (-):** A coloured line appears at the control region (C) and a coloured line appears at a specific drug test region (T). The appearance of a control line and test line indicates a negative test result. The test lines may have varying intensity either weaker or stronger in colour than that of the control line.

**Positive (+):** A coloured line appears in the control region and no coloured line appears at a specific drug test region. The complete absence of a test line indicates a preliminary positive result for that particular drug. A preliminary positive result for a drug indicates that the concentration of that drug in the urine at or above the cutoff level.


**Invalid:** No coloured line appears in the control region. If the control line does not form, the test result is inconclusive and should be repeated.

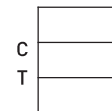

Negative (-)



Positive (+)

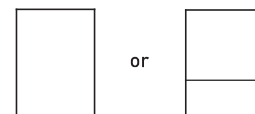

INVALID

Sober Check - The leader in certified drug & alcohol testing equipment

call 0800 700 777  
visit [sobercheck.co.nz](http://sobercheck.co.nz)